



iPhone and EufyLife

Download the EufyLife App

1. Access the Apple app store.
2. Click the search icon and enter “EufyLife.”
3. Download the EufyLife app by selecting “Get.”
4. When the app is fully downloaded, select “open” to create a profile.

EufyLife Sign-up

1. When open, iPhone will ask for the app to have permission to send notifications. Select “allow.”
2. The EufyLife app will display the End User License Agreement. After reading the agreement, select “accept.”
3. The app will then present three slides of information on app functions. To view each slide swipe your screen from left to right. On the last slide, select “experience now.”
4. The next screen will prompt the user to “log in” or “sign up.” If you are a first time EufyLife app user, you will select “sign up”. If you already have a EufyLife account, you will select “log in.”

MyNukaConnect

support@mynukaconnect.com

<https://www.mynukaconnect.com/>



5. To create an account, you will verify that your location is accurate. This will auto populate for you. Then enter your email address and create a password.
If you have an EufyLife account, you can skip to pairing your device and connecting to Apple Health.
6. The app will then prompt you to enter demographic information such as your gender, name, age, and height.

EufyLife Device Pairing

Note: Prior to pairing your device

1. Place batteries in your scale and check that it turns on by placing it on the ground and applying pressure.
2. Check that the Bluetooth on your phone is turned on. You can do this by opening your settings app.

Option 1

1. After completing your log in or sign up, the EufyLife app will ask you to “add device.”
2. Next, iPhone with request for the app to use your Bluetooth, select “ok” to grant app access.
3. While the app is searching for your scale, make sure your scale is near your phone and turned on. You can turn on your scale by setting on a hard surface and applying pressure.

Option 2

1. Select the three-bar display at the top left corner of the home screen to view settings.
2. Select “add device.”
3. Next, iPhone will request for the app to use your Bluetooth, select “ok” to grant app access.
4. While the app is searching for your scale, make sure your scale is near your phone and turned on. You can turn on your scale by setting on a hard surface and applying pressure.

EufyLife Connect to Apple Health

1. Select the three-bar display at the top left corner of the home screen to view settings.
2. Select “Connect to Apple Health.”
3. Next, iPhone will request for the app to have access to Apple Health. Select “Ok.”
4. On the Health Access page “Turn All Categories On.”
5. When all categories for data show green, select “Allow” at the top right of the screen.